



## Instructions for installation of CEREC SW 4.6 via web installer


1. Download the web installer of CEREC SW 4.6 on [my.cerec.com](http://my.cerec.com)
2. Start the downloaded **“CEREC\_460\_Webinstall.exe”**.
3. Run through the setup.
4. Wait for a pop-up when you get the “Download initiated” message.

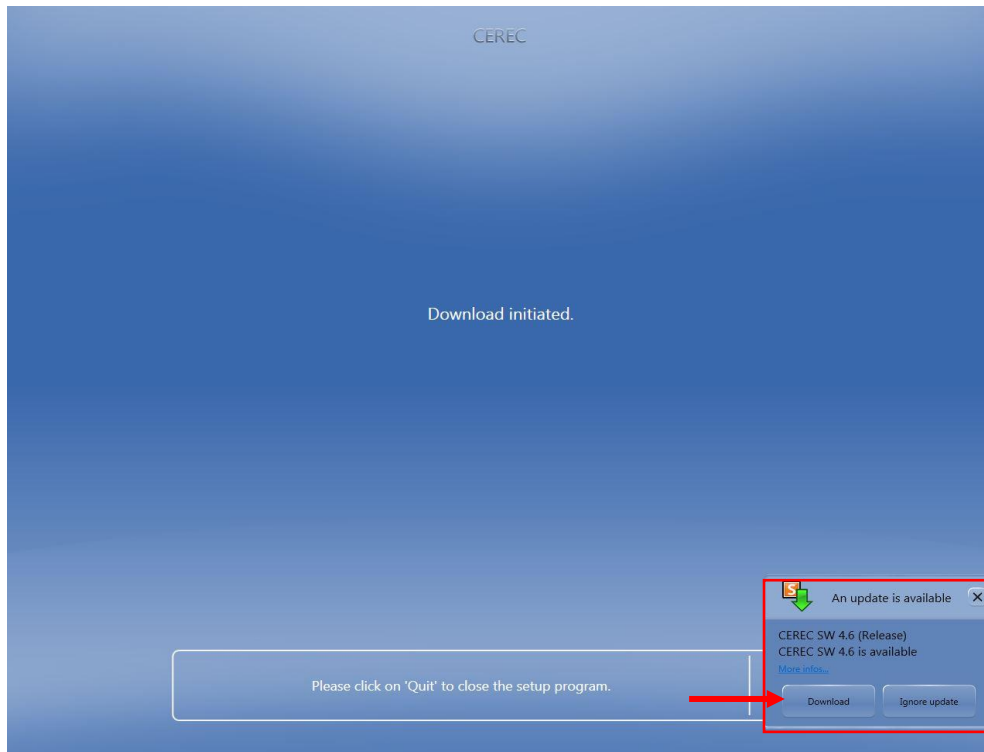

5. Select “Download” in the pop-up on the lower right. If you select “Ignore update” you will have to wait 24 hours to restart the AutoUpdate.
6. Wait until the **“CEREC SW 4.6 (Release)”** is downloaded (100%). If you stop the download you will have to wait 24 hours to restart the AutoUpdate. If the network is disconnected during downloading, the download continues automatically where it stopped.

AutoUpdate @ Sirona Connect Center			
Name	Description	More infos...	Download progress
CEREC Premium SW 4.5.0 MP (Release)	Material packet for CEREC Premium SW 4.5.0 is available.	<a href="http://my.cerec.com/de-de.html">http://my.cerec.com/de-de.html</a>	100%  
CEREC Premium CAM SW 4.5.0 MP (Release)	Material packet for CEREC Premium CAM SW 4.5.0 is available.	<a href="http://my.cerec.com/de-de.html">http://my.cerec.com/de-de.html</a>	100%  
CEREC SW 4.5.2 MP (Release)	Material packet for CEREC SW 4.5.2 is available.	<a href="http://my.cerec.com/de-de.html">http://my.cerec.com/de-de.html</a>	100%  
CEREC SW 4.6 (Release)	CEREC SW 4.6 is available	<a href="http://my.cerec.com/de-de.html">http://my.cerec.com/de-de.html</a>	0% 

7. Select **“Install”** on the pop-up on the lower right. If you select **“Don’t show again”** you have to wait 24 hours to restart the AutoUpdate.

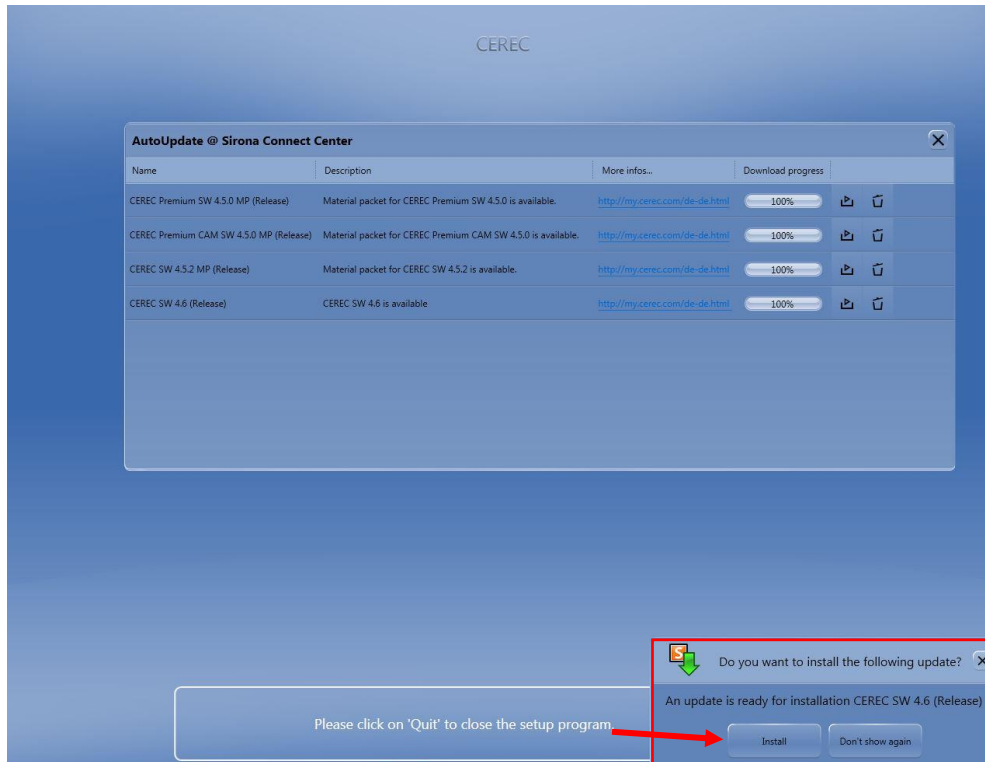

8. CEREC SW 4.6 Setup starts automatically.
9. Run through the CEREC SW 4.6 Setup.
10. *Optional: CEREC SW 4.5.x will be uninstalled. Don't check the boxes of deleting Patient Database, TLib Bio and Logs.*

*After uninstalling the CEREC SW 4.5.x the AC will restart automatically.*

*After the restart, the CEREC Setup needs to be restarted via the desktop icon "Continue Installation CEREC SW 4.6". This can be done at any time.*

*Run through the CEREC SW 4.6 Setup until CEREC SW 4.6 is installed.*

11. Enjoy your CEREC SW 4.6!